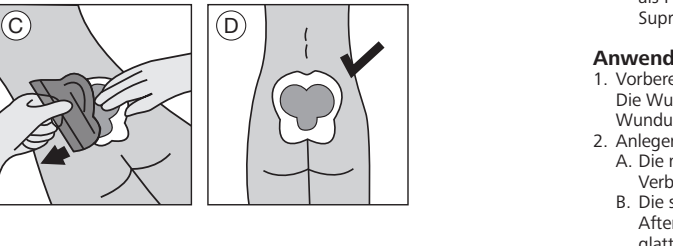
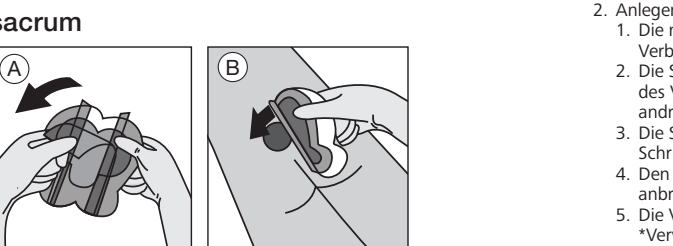
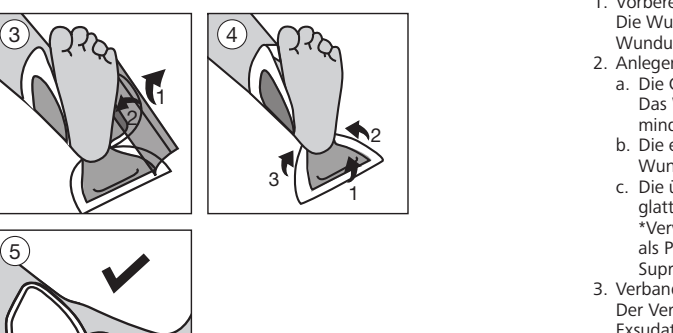
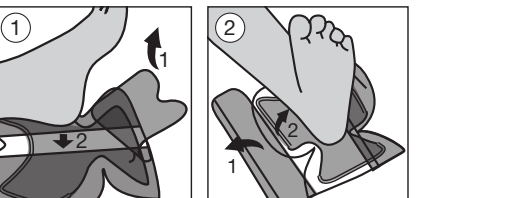
Suprasorb® P sensitive heel, sacrum, multisite border

Silikon-Schaumverband
Pansement hydrocollaïre siliconé
Silicone Foam Dressing
Apósito de espuma y silicona
Ligadura esponjosa de silicone
Medicazione in schiuma di silicone
Siliconen schuimverband
Silkone-schuimverband
Silikon-skuumbandage
Silikonvaahdosliina
Silikonový pěnový obvaz
Silikonový penový obváz
Silikon habótés
Opatrunek z pianki z warstwą silikonu
Obloga iz silikonске penne
Pligota sa silikonem pianom
Silikonlu köpük yara örtüsü
ضمادة السيليكون الحبيبية

heel

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Produktbeschreibung

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Instrucciones para Suprasorb® P sensitive sacrum:

1. Preparar the wound
Clean the wound in accordance with current clinical standards. Dry the surrounding skin thoroughly.
2. Applying the wound dressing
a. Remove the protective film from the middle section and position the adhesive side of the dressing on the skin.
b. Position the narrow side of the dressing at least 2 cm above the anal sphincter and smooth down the dressing without tension.
c. Remove the protective film from the sides and press down the dressing lightly. Please ensure that the wound dressing covers the entire wound area.
d. Smooth down the edges and check that there are no creases. The dressing can be repositioned if necessary.
*Where necessary, for deep wounds apply a rope as primary dressing (e.g. Suprasorb® A, Suprasorb® A + Ag, Suprasorb® Liqueur).

Precautions and warnings:

• The product is for single use only.
• Do not use it if the packaging is damaged or opened.
• Do not use Suprasorb® P sensitive Silicone Foam Dressing together with oxidising agents such as hypochlorite solutions or hydrogen peroxide.
• If reddening or sensitisation occur discontinue use and consult a health care professional for appropriate treatment.
• Because of the lack of data, cautious use is advised in sensitive population groups such as infants, children, pregnant or nursing women.

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en Instructions for use

Suprasorb® P sensitive border consists of a silicone wound contact layer, an absorbent polyurethane foam, a non-woven distributing water, a super-absorbent core and a protective, vapour permeable and waterproof backing. The five-layer construction facilitates dynamic fluid management to provide maximum optimal moist wound environment which leads to the promotion of faster wound closure and may help to reduce the risk of maceration. The gentle silicone layer can be lifted and repositioned without losing its adherence. Also, this layer can minimise pain to patients and trauma to wounds and the surrounding skin during dressing changes. The absorbent pad layer provides superior absorption and locks exudates away.

Product description

Suprasorb® P sensitive border consists of a silicone wound contact layer, an absorbent polyurethane foam, a non-woven distributing water, a super-absorbent core and a protective, vapour permeable and waterproof backing. The five-layer construction facilitates dynamic fluid management to provide maximum optimal moist wound environment which leads to the promotion of faster wound closure and may help to reduce the risk of maceration. The gentle silicone layer can be lifted and repositioned without losing its adherence. Also, this layer can minimise pain to patients and trauma to wounds and the surrounding skin during dressing changes. The absorbent pad layer provides superior absorption and locks exudates away.

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Product composition

Silicone, polyurethane, polyacrylate, fluff pulp and polyolefin

Intended purpose

Suprasorb® P sensitive Silicone Foam Dressing may be used to treat all wounds, provided they are:
• moderately to heavily exuding and
• superficial to deep.

Indications

• Pressure ulcers
• Leg and foot ulcers
• Traumatic wounds
• Surgical wounds
• Skin tears

The dressing may also be used as part of a prophylactic therapy to help prevent pressure ulcers.

Contraindications

Known allergy and/or hypersensitivity to any of the product components

Application instructions

May be applied by lay persons under supervision of health care professionals. The product can be used under compression bandaging.

1. Preparing the wound
Clean the wound in accordance with current clinical standards. Dry the surrounding skin thoroughly.
2. Applying the wound dressing
a. Remove the protective film from the middle section and position the adhesive side of the dressing on the wound. The wound should have a depth of at least 1-2 cm.
b. Remove the remaining release liner(s) and smooth down the border on the skin. Do not stretch the dressing.
*Where necessary, for deep wounds apply a rope as primary dressing (e.g. Suprasorb® A, Suprasorb® A + Ag, Suprasorb® Liqueur).

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