
Suprasorb® P sensitive non-border

Silikon-Schaumverband
Pansement hydrocellulaire silicone
Silicone Foam Dressing

Apósito de espuma y silicona
Ligadura esponjosa de silicone
Medicazione in schiuma di silicone
Siliconen schuimverband
Silikone-skumforbinding
Silikonskumförband
Silikon-skumbandasje
Silikonivaahtosidos
Silikonový pŕenový obväz
Silikónový penový obväz
Szlikon habkötés
Opatrunek z pianki z warstwą silikonu
Obloga iz silikonske pene
Povoj sa silikonskom pjenom
Αρωδέας σλικονούχο επίθεμα
Silikonlu köpük yara örtüsü

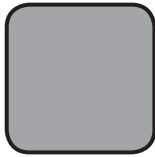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en Instructions for use

Product description:

Suprasorb® P sensitive non-border consists of a silicone wound contact layer, an absorbent PU foam and a protective backing. The triple-layer construction facilitates dynamic fluid management to provide an optimal moist wound environment, which promotes faster wound closure and may help to reduce the risk of maceration. The gentle silicone layer can be lifted and repositioned without losing its adherence. This layer can also minimise pain to patients and trauma to wounds and the surrounding skin during dressing changes. The polyurethane foam layer provides superior absorption and locks away exudate.

Indications:

Suprasorb® P sensitive Silicone Foam Dressing non-border may be used to treat all wounds, provided they are

moderately to exuding and

superficial to

deep*:

- Pressure ulcers
 - Leg and foot ulcers
 - Traumatic wounds
 - Surgical wounds
 - Donor sites
 - Skin tears
 - Burns (1st and 2nd degree)
- The dressing can also be used on dry/necrotic wounds in combination with gels.

Application instructions:

1. Clean the wound in accordance with normal procedures.
2. Dry the surrounding skin thoroughly.
3. Remove the release films and apply the adherent side to the wound.
*Where necessary, for deep wounds apply a wound dressing in accordance with the instructions that come with that dressing.
4. The dressing should overlap the dry surrounding skin by at least 1-2 cm for the smaller sizes and 5 cm for the larger sizes.

Warnings:

- This product is single use. Do not use it if the packaging is damaged or opened.
- Do not use Suprasorb® P sensitive Silicone Foam Dressing non-border together with oxidising agents such as hypochlorite solutions or hydrogen peroxide.
- If reddening or sensitisation occur discontinue use.
- Suprasorb® P sensitive Silicone Foam Dressing non-border can remain in situ up to 7 days, dependent on patient condition and the level of exudate. Do not exceed the wearing time of 30 days if worn continually.

Storage and disposal:

1. Please store the product in cool and dry place and avoid light.
2. Disposal should be handled according to local environmental procedures.

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